





# BD Nexiva™ Single Port Closed IV Cannula System


## Tips for Success

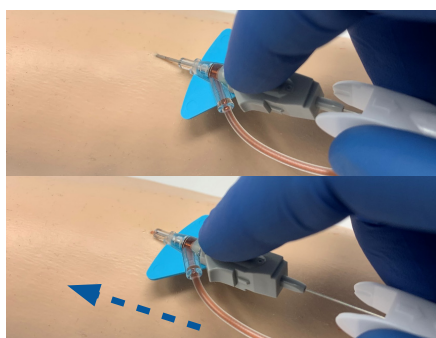

**1.** On Insertion, once the needle and catheter tip is in the vessel, advance the catheter off the needle guide using the grey push tab, DO NOT pull back on the white finger grips while pushing the catheter forward as this may result in a failed cannulation.

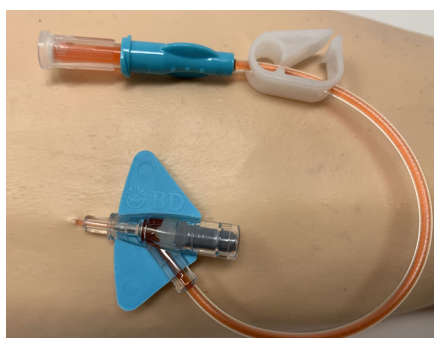

**2.** BD Nexiva™ Closed IV Catheter System self-primed with the patient's blood on insertion.

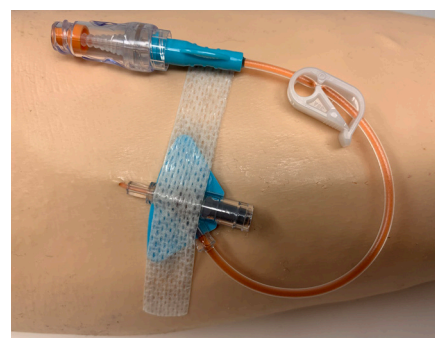

**3.** After insertion, secure the cannula, engage the clamp, remove the vent plug and attach the hospital's needle free IV access device.



**4.** When dressing BD Nexiva™, place the dressing off centre so the extension tubing exits through the gap, the stabilisation platform should be completely covered, and the insertion point visible.

Flush BD Nexiva™ using a push-pause technique, clamping on positive pressure to maintain patency. **Refer to specific needle free device instructions for use for specific clamping sequence.**

The clamp should be engaged, and the fluid pathway should be clear after flushing and when the cannula is not in use.

**Flow rates for BD Nexiva are as follows:**

Colour	Yellow	Blue	Pink	Green	
Gauge	24G	22G	20G	18G	
Length	19	25	25	32	
Flow Rate	ml/min	18	33	61	84
	ml/hour	1,080	1,980	3,660	5,040
Product code	383511	383512	383516	383519	

### Durbin Ireland (Uniphar) contact details:

bdorders@durbinireland.ie  
+353 1 428 7882

Durbin Ireland (Uniphar), Unit 645, Jordanstown Drive, Greenogue Business park, Rathcoole, Dublin 24, D24 YD83

[bd.com](http://bd.com)

BD, the BD Logo and Nexiva are trademarks of Becton, Dickinson and Company. © 2020 BD. All rights reserved. (BD-17796 Sept 2020).

