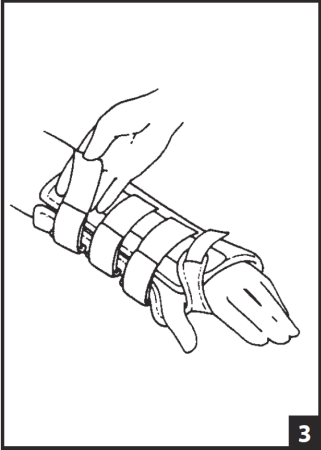
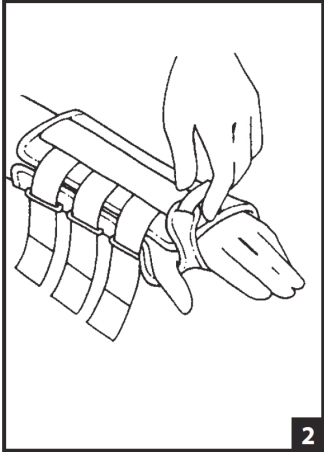
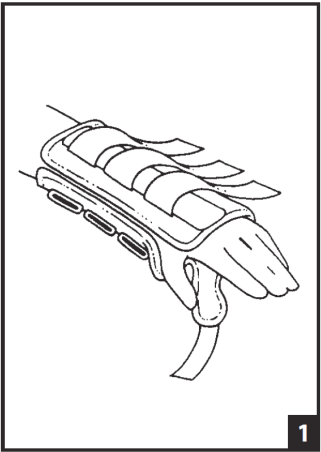
ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE:
The intended user should be a licensed medical professional, the patient or the patient's caregiver. The user should be able to read, understand and be physically capable of performing the directions, warnings and cautions in the information for use.

- INTENDED USE/INDICATIONS:** Post surgical immobilization • Post fracture casting • Wrist sprains/strains.
- A. Place hand and wrist into splint with thumb in slot as shown. (Fig. 1)
- B. Palmar and dorsal stay should align with arm. Dorsal stay/strap assembly may be repositioned for proper alignment.
- C. Secure straps starting at the thumb (Fig. 2) and continue up the arm (Fig. 3). Straps should be snug but not constrict.

- CONTRAINDICATIONS:** None
- USE AND CARE**
- Hand wash in cold water using mild soap, rinse thoroughly. AIR DRY. **NOTE:** If not rinsed thoroughly, residual soap may cause skin irritation and deterioration of the material.
 - The aluminum stays are malleable and can be molded for specific requirements.
 - Excess strap material may be trimmed.

WARRANTY: DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

- WARNINGS AND PRECAUTIONS:**
- If you have skin disorders/injuries in the relevant part of the body, the product should be fitted and worn only after consultation with a medical professional.
 - If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately.
 - Do not use this device if it was damaged and/or packaging has been opened.

FOR SINGLE PATIENT USE ONLY.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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©2021 DJO, LLC