



AfterSwim[®] Water Removal

AfterSwim is intended to remove water from the ear canal quickly and safely.

Reorder #530

Contents:

20 AfterSwim[®] Water
Removal Sponges



bionix[®]

**Health
at Home**

PROFESSIONAL PRODUCTS FOR HOME USE

MADE IN
THE U.S.A.



©2021 Bionix Health at Home
call 800.551.7096 | fax 800.455.5678
email bionix@bionix.com
visit 1670 Indian Wood Circle, Maumee, OH 43537
shop www.bionix.com | Patent# 8,784,352
RM95-5051 | Rev. D

AfterSwim® introduces a new approach to drying the ear canal in an effort to remove water from clogged ears or simply maintain cleanliness.

DRY



WET



- Flared design eliminates the risk of over insertion.
- AfterSwim® only needs to be inserted far enough into the ear to break the surface level of the water.

Instructions for Use

1. Remove a single blue AfterSwim® from its package.
2. Insert one of the ends of AfterSwim into the ear and hold it in place for 10 seconds.
3. Remove the AfterSwim from the ear. If water is still in the ear, rotate the AfterSwim and insert the dry tip into the ear.
4. Hold the AfterSwim in the ear for 10 seconds.
5. Remove the AfterSwim from the ear and discard.
6. If water remains in the ear, repeat the process with a new AfterSwim until the water is removed.

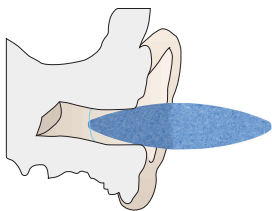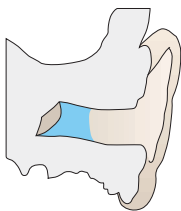

Indications for Use

AfterSwim is used for removing residual water from the ear canal after swimming or bathing.

WARNINGS/CAUTIONS

If ear pain occurs, please see your physician.