

ShotBlocker[®] User Instructions

INTENDED USE:

ShotBlocker is intended to distract, comfort, and reduce the anxiety of a patient receiving minor injections and/or needle pokes.

INDICATIONS FOR USE:

ShotBlocker is designed to distract and comfort patients receiving intra-muscular or subcutaneous injections and needle pokes.

WARNINGS/CAUTIONS:

Intended to be used for single user. **DO NOT** share between users.

INSTRUCTIONS:

1. Remove one ShotBlocker from its package.
2. Select the injection site and prep the skin as instructed by your medical professional.
3. Press ShotBlocker firmly against the skin so that the blunt contact points touch the skin at the injection site.
4. Immediately administer the injection in the usual manner through the central opening of ShotBlocker.

NOTE:

- a. Do not move or remove ShotBlocker until the injection has been completed.
 - b. If more than 20 seconds elapse between the placement of ShotBlocker and the injection, completely remove ShotBlocker from the skin and repeat the process beginning with Step 3.
5. Dispose of the needle and syringe in the proper container.
 6. Clean ShotBlocker with mild detergent and water and allow to dry.

CLEANING INSTRUCTIONS:

After each use, clean ShotBlocker with mild detergent and water. Dry completely.



MADE IN
THE U.S.A.



Obelis s.a • Boulevard Général Wahis 53 • 1030 Brussels, BELGIUM
Tel: +(32) 2.732.59.54 • Fax: +(32) 2.732.60.03 • E-Mail: mail@obelis.net
Bionix LLC • 1670 Indian Wood Circle, Maumee, Ohio 43537, United States
Phone: +1 419.727.8421 • Fax: +1 419.727.8426 • www.Bionix.com