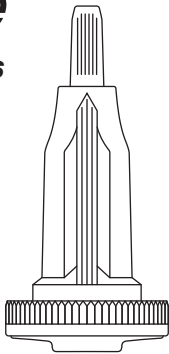
DropSafe™
Haemolance® Plus
Safety Lancets

Available in six sizes
Disponible en seis tamaños
Disponible en six formats

Blade Cuchilla X Penetration depth Profundidad
Bistouri Profondeur

1.5 mm x 1.6 mm
1.5 mm x 1.2 mm

Needle gauge Calibre de la aguja X Penetration depth Profundidad
Calibre de l'aiguille Profondeur

28G x 1.6 mm
25G x 1.4 mm
21G x 1.8 mm
18G x 1.8 mm

Safety Lancets

Sterile, single use medical devices intended for capillary blood sampling.
Button activated safety lancet. After use, the needle automatically retracts into the housing, protecting user from accidental needlestick injuries.

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Immunization Safety Suite

Capillary Blood Sampling

DropSafe provides innovative products that prioritize and enhance the safety of healthcare professionals and home users. For more information on the full safety portfolio, visit www.DropSafe.info or check with your local wholesaler.

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Symbols glossary:
Glosario de símbolos:
Glossaire des symboles:

References obtained from ISO 15223-1 Medical devices — Symbols to be used with medical device labels, labeling, and information to be supplied.

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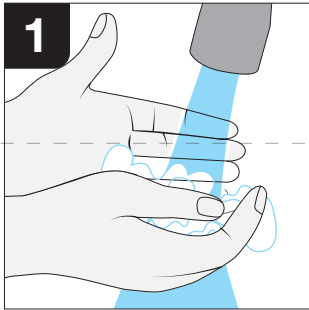
Instructions for use / Instrucciones de uso / Mode d'emploi

! Caution:

For use on a single patient, do not use on more than one patient. Not intended for more than one use. Improper use of blood lancets can increase the risk of inadvertent transmission of bloodborne pathogens, particularly in settings where multiple patients are tested. Protective cap is a single sterile barrier system. Do not use the safety lancet if the protective cap has been previously removed from the device. Risk of injury. If the penetration depth is too deep, the puncture can injure children. Before using the lancet for the first time on children, ask your healthcare professional. Start with the lowest penetration depth. Discard the entire used safety lancet after use into a sharps container in accordance with facility guidelines and local regulations.


Please consult with a healthcare professional which size of the device to use for required test blood volume.

Wash hands with warm water and soap. Dry thoroughly. Disinfect using an alcohol prep pad, ensure the finger is completely dry prior to lancing. Healthcare professionals, wear gloves prior to disinfecting the lancing site and completing the lancing procedure following the facility's protocol.

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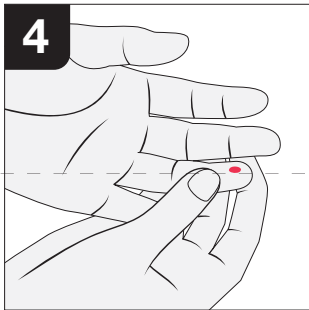Gently apply intermittent pressure near the puncture site to obtain the required blood volume.

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
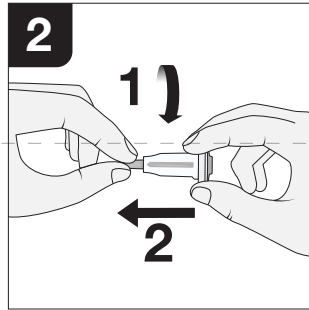
Twist off the protective cap and pull the protective cap straight out.

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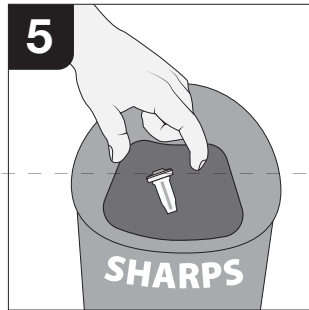Dispose of the entire safety lancet after use into an approved sharps container in accordance with facility guidelines and local regulations.

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
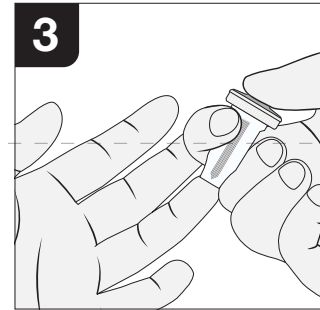
Press the safety lancet body firmly against the puncture site and push the button to activate the device. Avoid the finger pads, lancing at the side or top of the finger.

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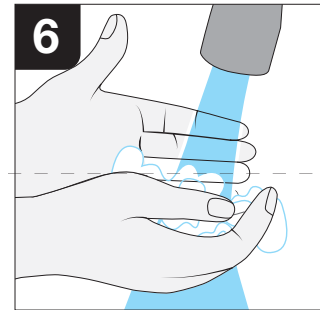The user should wash hands with warm water and soap after safe disposal of the safety lancet.

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