



rhythmhc.com | contactus@rhythmhc.com | (877) 843-6464

Please read and understand instructions
before using the charger.

The external battery charger is designed to power the P2
battery. Use the External Charger and P2 as directed.

The battery can be charged with the portable charger,
AC Adapter or DC Adapter.



2.5
Hours
to fully charge



1. Connect one end of the AC power cord to a power source and the other end to the external charger.



2. Slide the battery into the charger until you hear it click into place.



3. Flashing green lights (at the top of the charger) indicate that the battery is charging.

4. When all five green lights are solid, the battery is fully charged.



5. Disconnect from power. Press down on the latch on the bottom of the P2 and slide the battery out of the charger.

IMPORTANT!

Use Power port #1 for PAK3 AC Adapter

Use Power port #2 for P2 AC Adapter

